

EZ-Clean Manifold Retrofit Kit Instructions for Use

NOTE: It is recommended that you use a pouch without a gas filter:

1. Cut V channel at top of pouch as shown in Figure 1.



V cut at top of pouch Figure 1.

2. Insert manifold in pouch through wafer connection opening (2 piece pouch) or from drain opening shown for one piece pouches. Work manifold in position to force junction barb at center through V cut so it protrudes out top of the pouch Figure 2.



Figure 2. Insert Manifold in pouch until white Barb protrudes through V cut in pouch

3. Fold pouch border down at V cut and apply a layer of adhesive glue around the Luer barb see Figure 3. Apply glue at all connections to lock together.



Figure 3. Application of glue around base of barb

4. Insert Cap / tether as shown in Figure 4. and secure by pushing joint hose firmly onto the Luer connection sticking out the top of the pouch see Figure 4. Allow ½ hour for glue to cure before using the pouch.



Figure 4. Insert tether with Cap



Figure 5. Apply clear plastic joint hose securely to pouch



Figure 6. Manifold pouch retrofit installation complete, allow ½ hour for drying

Web Site: www.ostomyezclean.com