

## Installation Instructions for Portable Water Feed System

The portable water feed hook-up installation should take approximately 5 minutes following the steps outlined below:

Patent Pending / All Rights Reserved / TM

**Step 1:** Remove the filter screen from the faucet using a pair of pliers. Check the size and determine if it is a female or male thread and match it up with one of the adapters supplied.



Faucet



(1) Loosen Filter Screen



Remove Filter Screen

**Step 2:** Attach the adapter with **washer in place** to prevent leakage and secure with outlet facing in the direction you prefer for convenience. Carefully insert the ¼ inch hose into the quick connect outlet fitting taking care to push it all the way in so that it seats securely. Pull on the hose to confirm that it has seated firmly.



### ***Warning***

**Always turn off the Faucet after each use of the Water Feed System.**

The Water Feed System should not be left under continuous pressure when not in use, because the internal parts or hose could malfunction and rupture causing water damage to your home.

(2) Attach Portable Water Feed Adapter to the faucet with seal washer in place (shipped on valve handle).

**Step 3:** Attach the other end of the ¼ inch hose to the Water Feed Wand by inserting the hose into the quick connect fitting at the end of the wand; pull on hose to confirm that it has seated firmly.

**CAUTION: Leak test before using the system**

**Enjoy the EZ Clean System Ease and Life Normality it Brings!**

**Step 4:** Remove the cover from the water closet and slip the holder clip onto the side of the water closet. Attach the Water Feed Wand to the clip. Replace the cover to the water closet and you have completed the installation.



**Portable System**

**Commode Hanger**

**Schena Ostomy Technologies, Inc.**

Revision 8/01/08

**2313 Harrier Run, Naples, Florida 34105, Tel: 239 263-9957**