


Stoma Surround Sealant Application Instructions:

1. Cut the wafer opening to the required stoma size and position the pouch on clean flat surface with wafer facing up.
2. Apply a thin layer of Stoma Surround Sealant on the wafer around the opening cut to fit the stoma. This can be accomplished by squeezing out a 1 ½ inch length of Sealant and then touching the end to a starting point on the wafer and then stretching Sealant to form a ¼ inch diameter bead, then carefully laying the sealant around the opening or in a U shape as shown :
3. Using a hair dryer on low, heat the skin surrounding the stoma and the skin barrier wafer after the Stoma Sealant has been applied for best results.

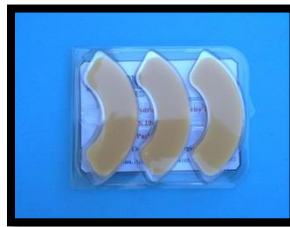

4. The skin should be prepped and read to receive the wafer application to the abdomen. Position the bottom half under the stoma and apply the pouch to the abdomen moving hands around all surfaces of the wafer to insure good adhesive contact with the stomach skin..
5. Use of the Hydrocolloid Skin Barrier Security Tabs is recommended to achieve maximum pouch wear time. The tabs can be applied to the inside or outside of the skin barrier wafer.



This completes the application process. For best results it is recommended that you wait 1 hour before cleaning the pouch after it has been applied to the abdomen to allow the Sealant to cure.



Schena Ostomy Technologies, Inc., 2313 Harrier Run, Naples, Florida 34105

Tel: 239 263-9957, web: www.ostomyezclean.com