

## Water Bottle Adapter Instructions

Schena Ostomy Technologies in their continuing effort to make life easier for ostomates has designed an adapter that permits portability for user's of the E-Z Clean pouch. This adapter allows most bottled water to provide a portable source of fluid to clean the pouch any place there is a bathroom. The adapter plugs into the top of the most water bottles and has a luer connection to mate with the pouch. A one way valve prevents back flow as the user squeezes the bottle to provide pressurized cleaning fluid to the pouch. Deodorizing, cleaning and lubricating agents can also be added to the bottle for increased efficiency. Small enough to be carried in a pocket or purse, this product provides the ultimate in convenience to a user.



**Water Bottle Adapter**



**Use Demonstration**



**Water Bottle Adapter Attached  
To E-Z Clean Pouch**

### **Use Instructions:**

- 1. Insert water bottle adapter in 6 or 8 ounce plastic bottle of water.**
- 2. Attach adapter Luer connection to the pouch inlet with  $\frac{3}{4}$  turn.**
- 3. Turn water bottle to vertical position and squeeze bottle to create cleaning pressure.**
- 4. Refill bottle as necessary to clean the pouch; replace pouch inlet cap to complete process.**



**2313 Harrier Run, Naples, Florida 34105**

**Tel: 239 263-995**

**Email: [ostomyezclean@comcast.net](mailto:ostomyezclean@comcast.net)**

**Web Site: [www.ostomyezclean.com](http://www.ostomyezclean.com)**