



# Schna Ostomy Technologies, Inc.

2313 Harrier Run

Naples, Florida 34105 Tel: 239 263-9957

## Ostomy EZ-Clean Product Data Sheet

8/01/2008

### **System:**

The EZ-Clean is a system designed to make life easier for those with an Ostomy to service a Colostomy or Ileostomy. The system uses liquid (water) under pressure to clean the inside of the pouch and exposed surfaces of the stoma in less than 3 minutes. The water feed system is a one time purchase for use with replacement pouches purchased as needed. A Water Feed Adapter is available for use with plastic container bottled water to clean the pouch when no utility water source is available.

### **Technical:**

The EZ-Clean pouch has a manifold built into the top interior to disperse the water, under pressure, to totally cleanse the interior of the pouch and the exposed stoma area. There is an inlet at the top of the pouch to permit easy hook-up of the water feed system (Portable or Fixed). The pouch is a one piece design with the wafer permanently attached. This is possible because of the ability of the user to thoroughly clean the pouch while still attached to the body, sitting on a commode (toilet). Evacuated contents are deposited in the toilet and simply flushed away without any personal contact.

When the EZ-System is not being used, the pouch can function like any other drainable pouch with the added advantage of the user being able to easily vent any accumulated gas through the inlet. It is important to note this characteristic because anyone concerned about becoming home bound by this system should realize this pouch can function like any drainable without using EZ-Clean. The system can operate in arrange of 20 to 80 lbs of water pressure. Below range pressures decrease efficiency.

### **Features:**

The pouch comes in a clear and opaque material and they have a comfort layer on both sides. The wafer is a space age medical foam composite for comfort and security and presently come as cut to fit only. Normally, no paste or seals are required with this product assuming normal application conditions. A convex product is available for those requiring it.

The wafer and pouch are bonded together to form a one piece system made possible because of the design which permits the user to hygienically clean the pouch interior without removal from the body. All pouches come with a docking tab which permits the user to fold the pouch in half and secure it with hook and loop material for user comfort and convenience. The pouch will release to full length if there is sufficient waste content that requires the volume.

### **Wear Period:**

The EZ-Clean Pouch was designed to last 3 to 5 days wear if properly attached to the wearer's abdomen. Proper preparation of the abdomen to accept the wafer is the key to successful wear time and will vary with the skin conditions of the user. The product is warranted to be free from defects in material and workmanship limited to replacement of the product found to be defective

### **Water Feed Systems:**

Currently there are two (2) water feed systems available: one is a portable system that can be easily attached to any bathroom sink ( hotel / motel included ) for travel: The second is fixed system that is semi permanently

attached to the water closet plumbing of a commode. Both systems can be installed by any person with minimal mechanical skills and come with a one (1) year warrantee. **WARNING: The water Feed Systems are connected to the main water supply for your home and must be shut-off after each use to prevent damage from any part failure.**