

Hydrocolloid Application to EZ-Clean Pouch Wafer

1. Lay the EZ-Clean Pouch on a clean flat surface and with the wafer facing up and remove the backing from the pouch wafer exposing the adhesive foam surface.



2. Take the Hydrocolloid wafer holding it between your fingers on two sides with the wafer cut instructions facing you (smooth side facing the pouch wafer). Line up the center ½” pilot holes and apply the Hydrocolloid wafer to the EZ-Clean Pouch wafer. The removal tab on the Hydrocolloid wafer should be at the 12:00 position to facilitate easy removal.



3. Using your fingers, rub the entire mating surface of the Hydrocolloid to insure a good bond between the pouch wafer and the Hydrocolloid surfaces. You can now cut the hole to your stoma size and both surfaces will be done at the same time.

The Hydrocolloid Skin Barrier is designed to provide secure attachment to the skin and will provide a barrier between the skin and pouch that is friendly to sensitive skin patients and has been made to complement the EZ-Clean Pouch System.